

Water Bottle Quick Release Conversion Kit

Installation Instructions

1. Turn off air pressure and remove existing bottle from manifold assembly.
2. Remove existing pick-up tube using care to not damage barb fitting. The kit comes with two blue replacement tubes, the short is for 750ml bottles and the long is for 2 liter bottles. Install the appropriate new tube.
3. Install male quick release into manifold and female quick release onto bottle and tighten both securely.
4. Install the bottle by simply pushing upward. The quick release will lock into place automatically; there is no need to push the button.
5. Turn on air pressure and listen for air leaks.

Operating Instructions

1. Turn off air pressure to bottle.
2. While securely holding the bottle, slowly depress the release button until you hear the air pressure exhaust. Hold release button at half way position until air pressure bleeds off. Fully depress the release button and remove the empty bottle.
3. Install a filled bottle by pushing upward until it clicks into place.
4. Turn on air pressure to bottle.

Universal, fits all bottles...
Only \$70 - While supplies last!



Save time and money by changing water bottles in seconds. And no more misaligned bottles!